




STARMed Ventukit

Hood for CPAP therapy with built-in Venturi flow driver

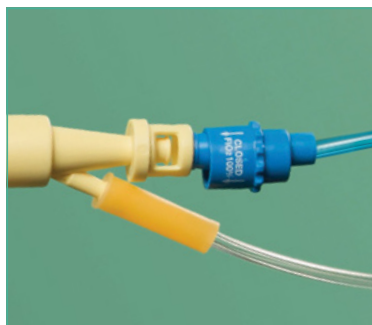

Critical Care ▪ Non-invasive ventilation ▪ STARMed



STARMed Ventukit CPAP hood with Venturi flow driver

StarMed Ventukit

StarMed Ventukit is a complete and easy to use system simply activated by connecting to the O₂ source available in the hospital departments. StarMed Ventukit unique design consists of oxygen tubes and a Venturi flow driver integral to a CPAP hood. The kit also includes an adjustable PEEP valve for immediate therapy activation.

Read more evidence on www.intersurgical.com

Technical Features

- The integrated Venturi flow driver can generate high patient flows while consuming a reduced amount of compressed oxygen
- O₂% can be adjusted from 40% to 100% at each PEEP level
- The adjustable PEEP valve can provide patient pressures in a range of settings up to 20 cmH₂O
- The user is guided by a performance table integral to the product, for independent PEEP and O₂% settings

Features and benefits

- Transparent for ease of patient monitoring
- Integrated pressure manometer
- Patient access port with bi-directional anti-asphyxiation valve



New design sealed access ports Closer to the patient's airway

Integral Venturi flow driver with oxygen supply tubes



Dual flow meter
Code: STS.15/30
Dual oxygen flow meter 15 + 30 L/min with O₂ coupling

Ventukit CPAP hood with Venturi flow driver, bi-directional anti-asphyxiation valve and patient access port, adjustable PEEP valve and ear plugs

Performance table subject to changes. Always refer to product labelling.

Requirement		Setting	Flow delivered
O ₂ % approx	PEEP cmH ₂ O	Flow O ₂ L/min A + B	Flow approx L/min
40%	5	9 + 0	46
	7.5	9 + 0	47
	10	10 + 0	48
	12.5	10 + 0	46
	15	11 + 0	47
	20	12 + 0	48
50%	5	8 + 9	47
	7.5	8 + 9	47
	10	9 + 9	45
	12.5	9 + 8	48
	15	10 + 8	50
	20	12 + 9	54

Requirement		Setting	Flow delivered
O ₂ % approx	PEEP cmH ₂ O	Flow O ₂ L/min A + B	Flow approx L/min
60%	5	7 + 16	52
	7.5	8 + 16	48
	10	8 + 14	48
	12.5	9 + 13	47
	15	9 + 13	47
	20	10 + 14	47
100%	Feed with a minimum O ₂ flow of 40 L/min		

Code	Size	Neck circumference	Box Qty.
CV100XXL/2	Extra Extra Large	≥50cm	5
CV100XL/2	Extra Large	45-52cm	5
CV100L/2	Large	40-47cm	5
CV100M/2	Medium	34-41cm	5
CV100S/2	Small	27-34cm	5
CV100XS/2	Extra Small	17-27cm	5



www.intersurgical.com/info/starmed

Make an enquiry



Intersurgical Ltd, Crane House, Molly Millars Lane, Wokingham, Berkshire, RG41 2RZ, UK
T: +44 (0)118 9656 300 F: +44 (0)118 9656 356 info@intersurgical.com www.intersurgical.com

Manufacturer: Intersurgical S.p.A. - Mirandola (MO) - Italy
The Intersurgical S.p.A. Organisation has a Quality Management System Certificate by KIWA CERMET ITALY S.p.A. according to ISO 9001 and ISO 13485

Please think before you print
Save energy and paper. If you must print this information sheet please print it double sided.

IS1.21_ROW • Issue 6 11.17